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Instructions for use of spectacle frames made of buffalo horn**REF**

These instructions apply to all Arte & Gifre eyeglass frames made from buffalo horn.

1. Introduction

Thank you very much for placing your trust in my work. You are now holding your new, custom-made buffalo horn glasses in your hands.

Even with my many years of experience in horn spectacle production, I have not been able to produce indestructible spectacles. Buffalo horn, mainly from Indian water buffalo, is a wonderful material in terms of its properties, but it is subject to a natural ageing process that can only be slowed down to a limited extent. The structure of horn absorbs moisture from the environment and the body and releases it again. The glasses expand and shrink. This interaction with moisture initiates the ageing process. The more extreme these conditions are, the faster the glasses age. Small cracks form in the horn and the surface slowly becomes dull and greyish. These are normal signs of wear and tear and are not indications of poor quality.

2. Intended use / purpose

The spectacle frame is designed to hold corrective lenses securely and should be worn in accordance with the general instructions for use for spectacle frames.

3. Contraindications, undesirable side effects

Buffalo horn is often used for spectacle frames and is generally well tolerated, especially as no metal parts come into direct contact with the skin. Nevertheless, attention should be paid to compatibility in the case of specific allergies.

4. Warnings

- Do not expose your buffalo horn glasses to unusual stresses, such as visits to the sauna, storage on the dashboard, working with strong cleaning agents and chemicals, increased sunlight or extensive sporting activities.
Be careful with impacts: the frame may deform in the event of a strong impact and cause injury.
- Long flights are rather problematic due to strong air conditioning. Tip for air travel: Place your horn-rimmed glasses in an airtight storage box, such as those found in every kitchen, for the flight, or use your spare glasses for the trip.
- Never clean your glasses with alcohol or solvent-based cleaning agents. These leach out the horn and can permanently damage your glasses. Perfume and cosmetics also often contain alcohol or chemical substances that can negatively affect the lifespan of your glasses.
- Allergies: Buffalo horn is generally well tolerated. However, if you have specific allergies, please pay attention to the material used.
- **Caution if you have a dog:** Dogs really like the taste of buffalo horn, so make sure your dog does not mistake your buffalo horn glasses for a chew toy.

5. Further information

- Customisation: As with all spectacle frames, your glasses should be adjusted by your optician to fit the shape of your face.
- Careful care and handling of glasses can significantly extend their life.

6. Cleaning

- It is best to use a clean microfibre cloth for cleaning. Make sure that there is no sand, dried lime or salt from the beach on the glasses. These substances can scratch your lenses.
- If there is still heavy dirt on the glasses, rinse them with lukewarm tap water and dry them immediately with a clean microfibre cloth.

7. Care

Horn-rimmed glasses age over time, so take them to an optician once a year for a service. The aged layer will be sanded down and the glasses sealed with a fresh polish. The longer you wait for the service, the more material will need to be sanded down.

8. Storage – protection from damage

- Store your glasses in a sturdy case when not in use. This will protect them from knocks, scratches and other damage.
- Make sure that the glasses are not squashed between other objects, as this can cause deformation, breakage or other damage.

9. Inspection and repair

- Check the frame regularly for any damage such as cracks, deformation or loose connections. If you notice any such problems, have your glasses checked by your optician immediately.
- If the frame is damaged, it is advisable to have it repaired by your optician. Do not attempt to repair the glasses yourself, as improper repairs may further damage the frame.

10. Disposal

If the glasses are damaged beyond repair, they should be disposed of properly (take them to a recycling centre or your optician).

11. Compliance

Our spectacle frames have been manufactured and tested in accordance with the EN ISO 12870:2018 standard "Spectacle frames – Requirements and test methods" and meet the requirements of the Medical Devices Regulation (EU) 2017/745. A corresponding EU declaration of conformity is available.

12. Contact for questions or uncertainties

If you have any questions or concerns about the product or its use, please ask your optician/manufacturer.

13. Reporting to the manufacturer and the competent authority

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is located.

Erläuterung der Symbole / Explication des symboles / Spiegazione dei simboli / Explanation of the symbols

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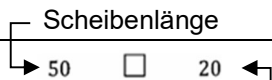
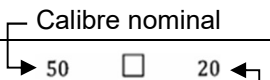
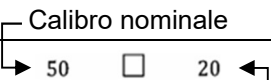
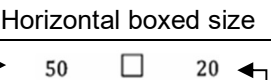
The symbols on the frame and on the packaging are explained below.

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Arte & Gifre AG